

## SOCK AID INSTRUCTIONS



1. Gather sock / stockings and pull over the end of the sock aid (end without straps).



2. Place the sock aid on the floor; hold the handles in an upward position away from the sock aid. Straps should be fully extended.



3. Slip one foot into the sock aid opening and pull upwards on the cords until the foot is completely inside the sock aid.



4. Continue to pull upwards on the handles until the sock is completely on the leg and the sock aid slides off.

### One Year Warranty

If this product proves to be defective in material or workmanship within one year of purchase, please return it to the address below. It will be repaired or replaced without charge upon receipt of the unit prepaid with \$5.00 to cover handling, packaging and return postage. Please include proof of purchase, your full name, address, daytime phone number or email address.

BIOS | Medical  
MANUFACTURED BY / FABRIQUÉ PAR :  
**THERMOR LTD.**  
16975 LESLIE STREET  
NEWMARKET, ON L3Y 9A1  
MADE IN TAIWAN / FABRIQUÉ À TAIWAN  
WWW.BIOSMEDICAL.COM